

PACS Upgrade

Olympic Radiology has upgraded our PACS (Picture Archive Communication System) to offer you more options and features for viewing patient reports and images.

ACTIVE X DOWNLOAD

Effective June 21st, Active X controls will need to be re-installed on a computer in order for the upgrade to be effective, and for you to continue to view images. This download will only need to be completed ONE TIME on a computer. If you have any issues with downloading Active X online, the files are available on CD. Please contact Allyson at (360) 616-0340 or Allyson@olympicradiology.com to request a copy.

ENHANCED PACS FEATURES

MORE COMPARISON OPTIONS


Previously, our PACS allowed the comparison of two studies at a time, where each study could be placed either in the left or the right half of available screen space. The new upgrade removes these limitations by allowing the display of series from ANY number of studies ANYWHERE, without hard split of the screen into the areas for a particular study.

UPDATED NAVIGATOR

Each thumbnail in the study Navigator displays the series description and indication of the current image in the series and the total number of images in the series. It also now indicates whether or not all images in the series have been viewed.

- All images in this series have been viewed.
- ▲ About half of the images in this series have been viewed.
- Less than 50% of images in this series have been viewed.

If more than one study is displayed in the Navigator, there are several attributes that you can customize to make the studies more easily identifiable. The size of Thumbnails in the study Navigator is also configurable.

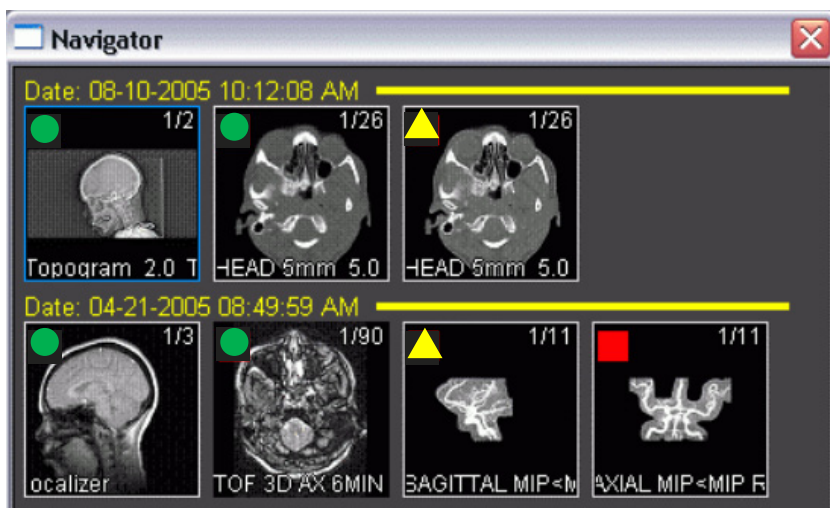

Right click on the navigator to view your options. For further instructions on how to customize, click on the Help menu, or contact Allyson at (360) 616-0340 or Allyson@olympicradiology.com.